**Inserting the lens**

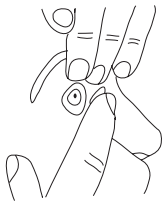
Fig. 1

**Removing the lens**

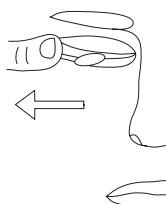
Fig. 2

**Centering the lens**

Fig. 3

Informatieblad / Feuille d'information / Information leaflet / Informationsblatt / Foglietto informativo

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**Patient information leaflet,**

to be provided to wearers of contact lenses by their contact lens practitioner

**Rigid gas permeable contact lens for use on the eye**

**Name of the product:** see packing label  
**Name of the material:** see packing label  
**Storage solution:** if present poloxamer 0.25%, EDTA 0.10%, and polyhexanide 0.0002% in a buffered isotonic and sterile solution.  
**Application:** suitable for use on the eye  
**Wear schedule:** in consultation with contact lens practitioner  
**Indication:** to correct refractive disorders, to optimise sight and cosmetic corrections  
**Replacement:** according to instructions contact lens practitioner

**Interactions with other products:**

- Only contact products which are recommended by the contact lens practitioner or which are related to your lenses, to avoid interactions between the contact lenses and the care products.
- Medications that are used over a long period of time may affect the wearability of the lenses. Be sure to inform your contact lens specialist of any medication you are taking.

**Not to be used in case:**

- Inflammation, infection, redness or irritation of the eye or eyelid.
- Change in cornea sensitivity or reduced tear production
- Any general disease that may affect the eye, or for which wearing contact lenses may increase the risk of complications.
- The contact lens is damaged

Consult your contact lens practitioner should any of these situations arise!

**Adverse reactions:**

Report any changes in eye comfort while wearing the contact lenses to your contact lens specialist; these changes may include increased tearing, severe redness, poor vision or allergic reaction. Permanent eye damage may occur if you do not report these changes.

**Inserting the lens: (Fig. 1)**

**Removing the lens: (Fig. 2)**

**Centering the lens: (Fig. 3)**

**If lenses are not being used for a longer period:**

- Clean the contact lenses thoroughly
- Keep the lenses dry in a clean and absolutely dry contact lens holder
- Clean the lenses immediately before you use them again
- Leave the lenses for 8 hours in a storage solution for a better comfort

**Special remarks:**

- When removing the lens, the contact lens must move freely on the eye and swim in the tear film. Therefore, use a moisturising liquid as recommended by your contact lens specialist and blink a few times until the lens moves freely when removing lenses.
- Consult your contact lens practitioner regularly to examine your contact lenses.
- The lenses and the contact lens holder should be regularly cleaned and looked after in accordance with the instructions provided by your contact lens practitioner. Do not use tap water to rinse the lens. Saline (sterile salt solution) is preferred.

**Inserting the lens**

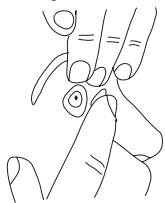
Fig. 1

**Removing the lens**

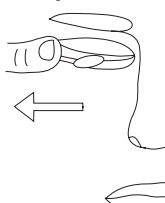
Fig. 2

**Centering the lens**

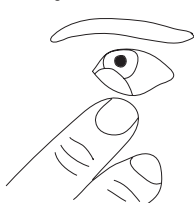
Fig. 3

Informatieblad / Feuille d'information / Information leaflet / Informationsblatt / Foglietto informativo

- Follow the accompanying instructions of the lens cleaning fluids and eye drops that you use accurately.
- Do not change your lens care system without consulting your contact lens practitioner.
- Do not exceed the wearing period of the lenses as recommended by the contact lens practitioner.
- When wearing rigid gas permeable contact lenses during swimming there is a chance of losing them!
- When using lenses for the first time, ensure that the packaging seal is not damaged or open. If this is not the case, contact the supplier.
- If the eyes are irritated as a result of being exposed to irritating vapours or fluids, take out the lenses and clean them thoroughly.
- Pregnancy or the use of oral contraception drugs may reduce the wearing comfort of contact lenses. Consult your contact lens practitioner if this occurs.
- Take care to have a spare set of lenses for replacement in case of your lenses get damaged or lost, and to guaranty you will always have a good vision in all situations.

**Bi- & Multifocal:**

- In low or poor light, bi-focal and multi-focal lenses may give unwanted reflections, which may interfere with normal vision. In particular, drivers are advised to check if their sight will be affected at twilight or dusk.

**DreamLite:**

- Application: reshape the cornea during the night.
- Before initial use, ensure that the specific instructions on use of the DreamLite lens have been explained by your contact lens practitioner.
  - During the adaptation time, annoying reflections can occur, which can cause a risk especially when driving at night. Therefore, DreamLite users need to be closely checked by a contact lens practitioner. DreamLite users are advised to check that they have undisturbed vision prior to participating in traffic.
  - The best guarantee to achieve a good and stable visual acuity, is to wear the lenses every night.
  - It can take some time before the best effect is achieved (up to 3 weeks), in particular for high minus powers.
  - Always follow the detailed instructions of your contact lens practitioner regarding insertion of the DreamLite lens.
  - Take great care of hygienic treatment of your lenses, to avoid complications
  - Follow the evaluation schedule established by your contact lens practitioner to ensure safe use of the DreamLite lens.
  - Replace the DreamLite lens on a yearly basis.
- Not following these instructions can result in unsafe use or ineffective results of the product.

**Expiry date:** See the package for expiration date. Do not use if the product has expired

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