Lens normal/ inside-out

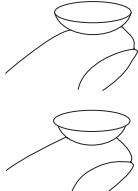
Fig. 1

Lens insertion

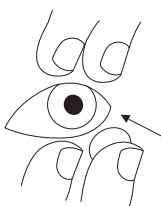
Fig. 2

Lens removal

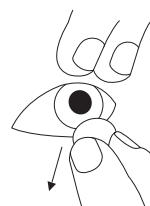
Fig. 3

Informatieblad / Information sheet / Informationsbrochure / Brochure d'informations / Foglietto informativo Folheto informativo / Folleto informativo

OP0175v7

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Information leaflet, to be provided to wearers of contact lenses by their contact lens specialist/ optometrist

Soft contact lenses for eye use

Product name: see packing label

Material name: see packing label

Storage solution: saline

Administration: suitable for eye use

Wearing schedule: the lenses are suitable for daily wear (cannot be worn overnight). Follow the wearing schedule provided by your contact lens specialist/optometrist.

Indication: to correct myopia refractive disorders and optimise vision ability

Sterility: lenses are sterilised by steam (sterility is only guaranteed in the closed and undamaged sealed package). Do not use the lenses if the packaging has been tampered with.

Replacement: according to instructions from the practitioner/optometrist

Interaction with other substances:

• To avoid unwanted interaction between the contact lenses and the cleaning solution, use only the cleaning solutions recommended by your contact lens specialist/optometrist. These solutions are compatible with your lenses.

• Medications that are used over a long period of time may affect the wearability of the lenses. Be sure to inform your contact lens specialist/optometrist of any medication you are taking.

Do not use when any of the following conditions exist:

- Inflammation, infection, redness or irritation of the eye or eyelid.
- Changes in corneal sensitivity or reduced tear production.
- Any general disease that may affect the eye, or for which wearing contact lenses may increase the risk of complications.
- Intolerance or abnormal eye reaction to the wearing of contact lenses
- The contact lenses are damaged.
- The wearer cannot or refuses to follow the instructions for safe contact lens use.

Consult your contact lens specialist/optometrist/ophthalmologist if any of these conditions occur!

Adverse reactions:

Report any changes in eye comfort while wearing the contact lenses to your contact lens specialist/optometrist, these changes may include increased tearing, severe redness, poor vision or allergic reaction. Permanent eye damage may occur if you do not report these changes.

Lens normal/inside-out (Fig.1): Lens insertion (Fig.2): Lens removal (Fig.3)

Special precautions:

- Visit your contact lens specialist/optometrist regularly to have your contact lenses checked.
- Wash your hands thoroughly with soap and dry them carefully before handling lenses.
- The contact lenses and the contact lens case must be regularly and carefully rinsed, maintained, and replaced according to the instructions provided by your contact lens specialist/optometrist. Do not use tap water to rinse the lenses. Saline (sterile salt solution) is preferred. Allow your lens case to air dry while open.
- Follow the instructions provided with the lens cleaning solution and eye drops closely.
- Do not change your lens cleaning solution without first consulting your contact lens specialist/optometrist.
- Do not wear the contact lenses for longer than the time recommended by your contact lens specialist/optometrist.
- In poor lighting conditions, bi- or multifocal lenses can create reflections that may impair vision. Ensure that your eyesight is unimpaired at dusk/night, particularly before operating a motorised vehicle.
- Users with a reading addition greater than +1.50 D should be carefully monitored by their contact lens practitioner during the adaptation phase. Visual acuity might be suboptimal during the adaptation phase.
- Do not wear soft lenses while swimming. This may lead to bacterial or viral infections.
- Do not use lenses that have been stored unused for a long period of time. In this case, use new lenses and discard the old lenses.
- When using lenses for the first time, ensure that the packaging seal is not damaged or open. If this is not the case, contact the supplier.
- If contact with smoke or liquids causes eye irritation, remove the lenses and rinse them thoroughly with saline.
- Pregnancy or the use of oral contraception drugs may reduce the wearing comfort of contact lenses. Consult your contact lens specialist/optometrist if this occurs.

Expiration date: See the package for expiration date. Do not use if the product has expired.

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